

3M™ Scotchbond™ Universal Plus Adhesive

The original, now better than ever!



The benchmark adhesive you trust...

3M™ Scotchbond™ Universal Plus Adhesive takes universal adhesives to the next level. It keeps all the benefits of the original 3M™ Scotchbond™ Universal Adhesive while adding even greater control and predictability – because a good adhesive should provide more than just a strong bond.

steps forward from the original

- Added radiopacity
- Bonds and seals caries-affected dentin
- Advanced bonding to dental and restorative substrates, including glass ceramics
- Full dual- and self-cure compatibility without additional dual-cure activator bottle
- Free of BPA derivatives like BisGMA






Plus, the original Scotchbond Universal Adhesive's popular features:

Universally suitable



⊘ Indirect

All etching techniques

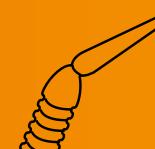

. now setting a new standard

Minimally Invasive Dentistry




Added radiopacity

Radiolucency under an existing restoration could lead to misdiagnosis and invasive overtreatment. 3M™ Scotchbond™ Universal Plus Adhesive is **the first radiopaque universal adhesive** with dentin-like radiopacity, helping to overcome this issue.

These doubtful X-rays happen every day; most often critical step in bitewing radiograph, class II in proximal box."

- Prof. Dr. Angelo Putignano, University of Marche-Ancona, Italy

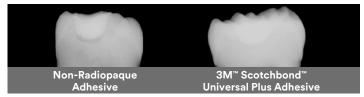

Intentional adhesive pooling under Class I restorations – X-rays of extracted human molars, filling material 3M™ Filtek™ One Bulk Fill Restorative. K. Dede et al., J Dent Res 99 (Spec Iss A): 192 IADR, 2020 – 3M internal data, H. Loll et al., J Dent Res 99 (Spec Iss A): No. 757, 2020


Radiolucent adhesive pooling underneath a restoration doubles the risk of restoration replacement."

- Dr. John O. Burgess, University of Alabama, USA


Bonds and seals caries-affected dentin to help preserve tooth structure

Scotchbond Universal Plus Adhesive gives you **confidence** when performing minimally invasive preparations because it has the same high bond strength to caries-affected dentin as to sound dentin.

In addition to the excellent quality of the restoration margins, it seals caries-affected dentin by forming a well-defined, void-free hybrid layer – because "the seal is the deal" to prevent secondary decay.

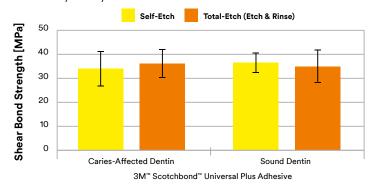

Shear bond strength on caries-affected and sound dentin – C. Thalacker, et al., Interaction of an Experimental Universal Adhesive with Caries-Affected Dentin, J Dent Res 99 (Spec Iss A): 191 IADR, 2020

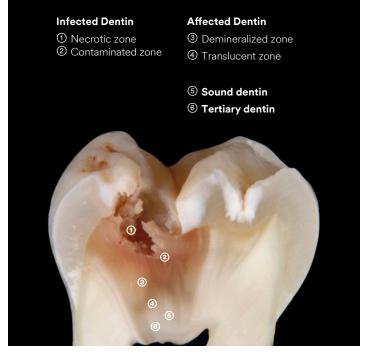

Caries zones – Image courtesy of Prof. Leandro Hilgert and Prof. Soraya Leal, University of Brasilia (UnB), Brazil

Bonds all restorative materials

without the need for separate primers


Virtually no post-operative sensitivity

99%

of dentists reported that **3M™ Scotchbond™ Universal Plus Adhesive** shows virtually no post-operative sensitivity*

Advanced bonding to glass ceramic

Scotchbond Universal Plus Adhesive reaches the gold standard level of traditional, separate silanes.

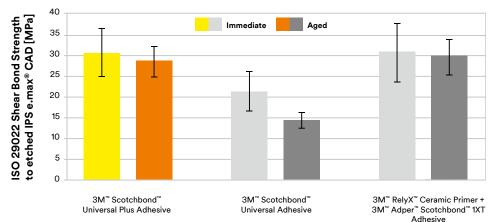

Shear bond strength on glass ceramic - Prof. Dr. Bart van Meerbeek et al., KU Leuven, Belgium, Report to 3M, 2020

Report conclusion:

3M[™] Scotchbond[™] Universal Plus Adhesive performed better than 3M[™] Scotchbond[™] Universal Adhesive for glass ceramic

(IPS e.max® CAD, Ivoclar Vivadent) bonding. After the glass ceramic was etched with HF acid, a favorable 'immediate' and 'aged' share bond strength were recorded.

Fully aligned system with 3M™ RelyX™ Universal Resin Cement

Excellent bond strength for virtually all dual-cure resin cement indications. Simplified workflows – adhesive cured by RelyX Universal Resin Cement.

Ordering Information

Item	no.	Product information
O N A™	Canta	ah han d™ Haiyayaal Dhu

3M'" Scotchbond'" Universal Plus Adhesive – Vial		
41293	Intro Kit Vial 1 Vial 5ml, 50 Disposable Brushes, 1 Etchant Syringe 3ml, 25 Dispensing Tips	
41294	Refill Vial – 1 × 5ml	
41295	Refill Vial – 3 × 5ml	
41263	3M [™] Scotchbond [™] Universal Etchant Etching Gel 2 Syringes x 3ml, 50 Dispensing Tips	

3M™ Scotchbond™ Universal Plus Adhesive – Unit Dose		
41297	Intro Kit Unit Dose 50 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips	
41298	Unit Dose 100 100 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips	
41299	Unit Dose 200 200 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips	
41304	Unit Dose 400 400 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips	



Vial

with smooth and even surface – easy to wipe off excess



for efficient hygiene management – reduces the risk of cross contamination



3M.com/ScotchbondPlus

3M Oral Care 2510 Conway Avenue St. Paul, MN 55144-1000 USA Phone 1-800-634-2249 3M.com/dental