Bluephase® EasyCure

Start-up

- 1

✓

Unpack handpiece

✓

Detach the light guide by pulling it out
- ✓
- Clean them (see Instructions for Use)

2

✓

Connect the power cord with the power supply

3

✓

Slide battery straight into the handpiece until you hear and feel it click into place.

4

✓

Place handpiece in the corresponding rest in the charging base

i


Disposable protective sleeves are recommended for each patient. Attach anti-glare cone or anti-glare shield to the light guide.

i

If the battery is fully discharged, the charging time is approx. 2 hours

5

Operation

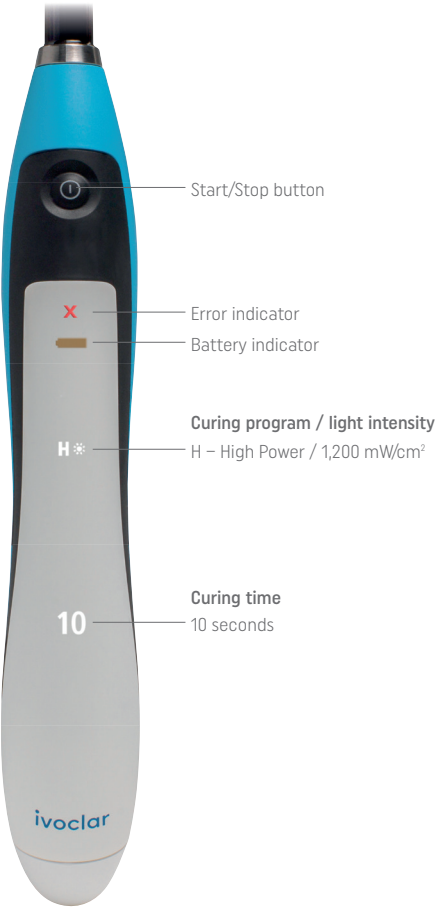What if ...?

If any of these errors persist after trying their corresponding rectification, please contact your dealer or your local Service Centre.

Indicator		Causes	Error rectification
Red "X" is flashing		The device is overheated.	Allow the device to cool down and try again after a certain time. If the error persists, please contact your dealer or your local Service Centre.
Red "x" lights permanently		Electronic component of the handpiece is defective.	Remove and reinsert the battery. If the error persists, please contact your dealer or your local Service Centre.
Red "x" lights permanently and the battery symbol is flashing		Battery empty	Place the device in the charging base and charge it.
		Battery contacts dirty	Remove battery and clean the battery contacts.
The charging base is not illuminated during charging		- Power cord not connected - Battery fully charged	Check if the charging base is connected to the power supply by means of the power cord.

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